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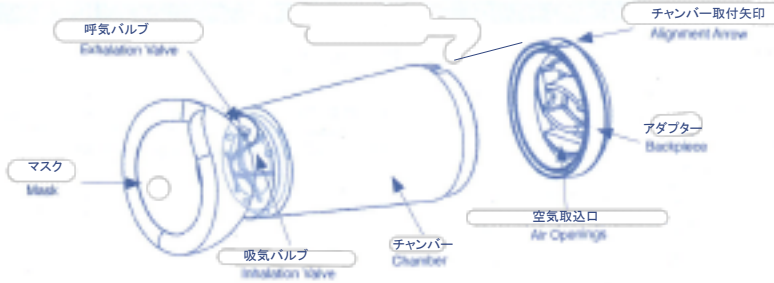
L'espace valved holding chamber (VHC) helps you use your metered dose inhaler (MDI). Thanks to its innovative inhalation valve and backpiece, L'espace helps the delivery of drug to your lungs and reduces undesired side effects.

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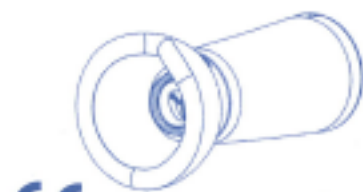
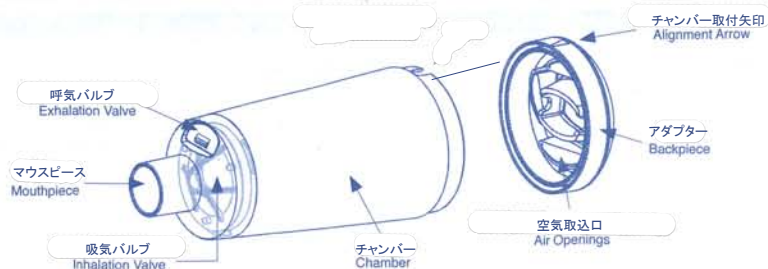
If the device is always used by the same person, clean it as shown.

If the device is used by more than one person, besides cleaning after each use, it needs to be sterilized in an autoclave at 121°C for at least 15 min. (preferably no longer than 30 min.). It can be sterilized without affecting the functional features up to a maximum of 20 times, after which it must be replaced. If needed, for a deep cleaning, it is possible to boil the device or to wash it in the dishwasher.

- Important:**
- The device should be replaced if any of the components gets damaged.
 - Some components are small enough to be swallowed by children, so they must never be left unattended. An adult should always be present.
 - Do not disassemble the device beyond what is recommended in the Cleaning Instructions.
 - If you are not sure on how to use the device, talk to your doctor or pharmacist.
 - To ensure efficient performance, the device must only be cleaned as instructed here.
- Notes:**
- Store in a clean container when not in use.
 - The device should be replaced after 24 months of continual use.
 - This product is latex-free.

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If the device is used by more than one person, besides cleaning after each use, it needs to be sterilized in an autoclave at 121°C for at least 15 min. (preferably no longer than 30 min.).

It can be sterilized without affecting the functional features up to a maximum of 20 times, after which it must be replaced.

If needed, for a deep cleaning, it is possible to boil the device or to wash it in the dishwasher.

Important:

- The device should be replaced if any of the components gets damaged.
- Some components are small enough to be swallowed by children, so they must never be left unattended. An adult should always be present.
- Do not disassemble the device beyond what is recommended in the Cleaning Instructions.
- If you are not sure on how to use the device, talk to your doctor or pharmacist.
- To ensure efficient performance, the device must only be cleaned as instructed here.

Notes:

- Store in a clean container when not in use.
- The device should be replaced after 24 months of continual use.
- This product is latex-free.

L'espace